
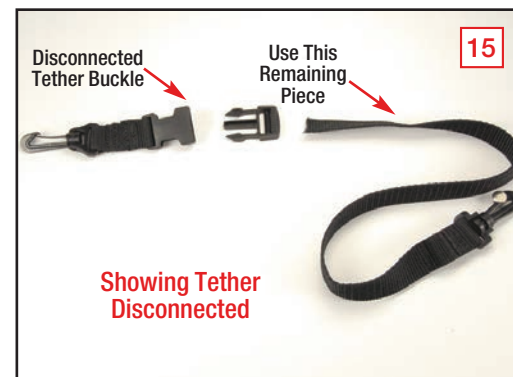
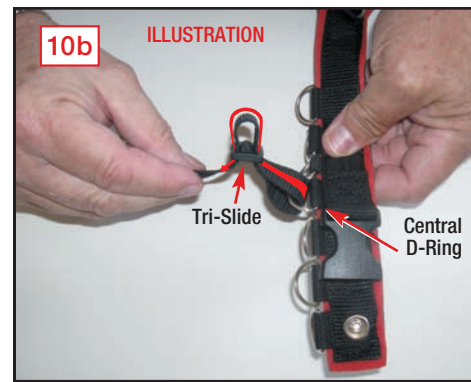


*For the Treatment
of Canine Cruciate Injuries*

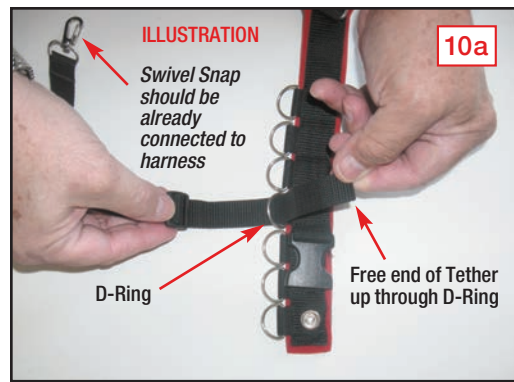

A-TraC Dynamic Brace®
Patent Pending

The Anti-Translational Cruciate Brace

by WoundWear, Inc.



- Conservative management
- Post-operative protection
- Rehabilitation



Please view our complete Guarantee online.

WOUNDWEAR, INC. ALLOWS EXCHANGES FOR SIZING ISSUES IF RECEIVED BY OUR COMPANY NO LONGER THEN 21 DAYS AFTER THE CUSTOMER RECEIVES THE PRODUCT.

WOUNDWEAR, INC. ALLOWS EXCHANGES FOR DEFECTS IN MATERIALS AND WORKMANSHIP IF RECEIVED BY WOUNDWEAR, INC. UP TO 30 DAYS AFTER THE CUSTOMER RECEIVES THE PRODUCT. DAMAGE TO THE PRODUCT MEANS EXCESSIVE USE AND ABUSE OR LACK OF CARE.

ABSOLUTELY NO REFUNDS ARE ACCEPTED FOR ANY REASON.

FOR ANY EXCHANGES PLEASE SEND PRODUCTS VIA TRACKABLE METHODS (i.e.; FedEx, UPS etc.) as we will require proof of delivery.

WOUNDWEAR, INC c/o IronLinx Distribution Services
578 Rosedale Rd.
Suite 27
Kennet Square, PA 19348

WOUNDWEAR, INC.
800-443-4055 (US and Canada only)
847-634-1700
woundwear@outlook.com











A-TraC Dynamic Brace® Design & Care

The materials of the A-TraC Dynamic Brace are soft and designed for your dog's optimum comfort. It is made with a two sided laminated Neoprene, which is soft but very durable. All areas are double to quadruple stitched where most stress occurs. The brace is designed to hold up to normal activity which is anticipated during the recuperation of a dog. Minimizing abuse of the brace by your dog such as biting, chewing, rubbing and excessive activity is very important in maximizing the braces longevity as well as success of treatment. In addition, the brace must be cared for every 7 to 10 days by cleaning the Hook and Loop Straps with a flea comb followed by using a sticky roller on the straps and the brace itself to remove hair. The brace should then be washed in cool water with mild detergent and then dried on low for 5 to 10 min. Close all brace and cuff straps to minimize lint sticking to it during wash.

Initial Testing of Components

Open, close and pull on all connections of the A-TraC Dynamic Brace components. If you find any defects or loose parts please send an e-mail to tech@woundwear.com with a close up picture of the defect from two different angles and follow with a phone call (800-443-4055) the same or next day. Notification of defects in materials or workmanship are required within 7 days of receipt of the brace. Decisions regarding defects in materials and workmanship after the 7 day time period will be reviewed in a case by case basis.

Illustration and Designation of Components for the A-TraC Dynamic Brace® Patent Pending

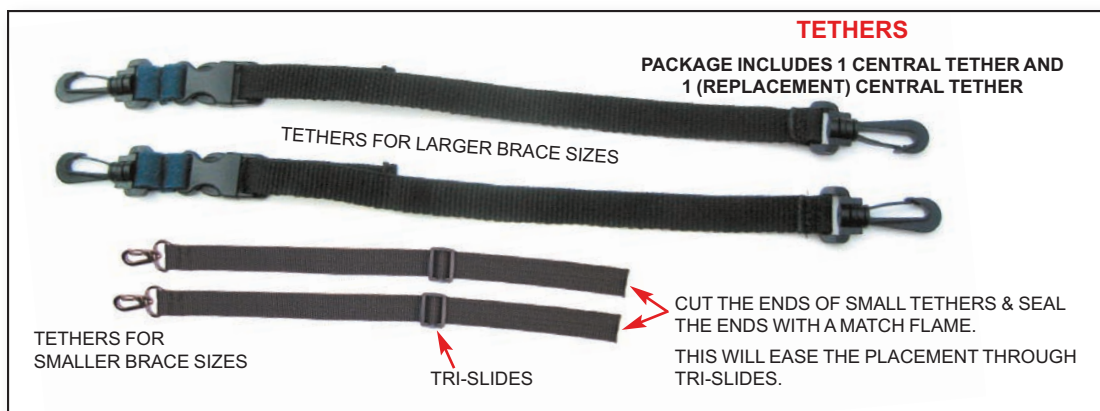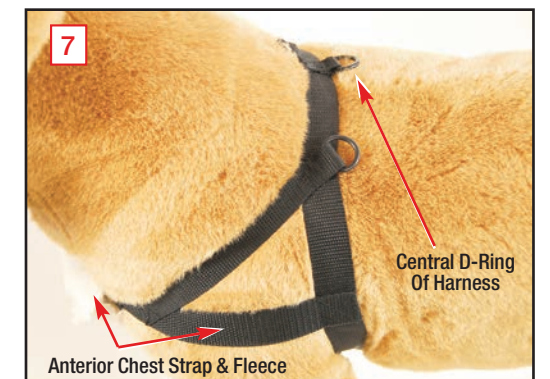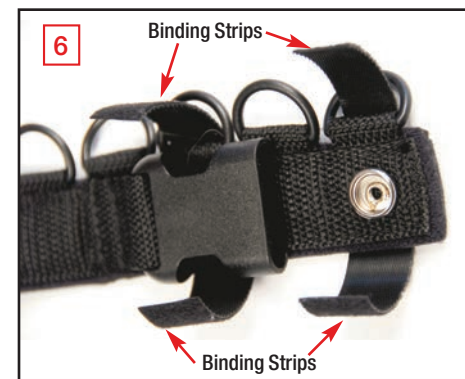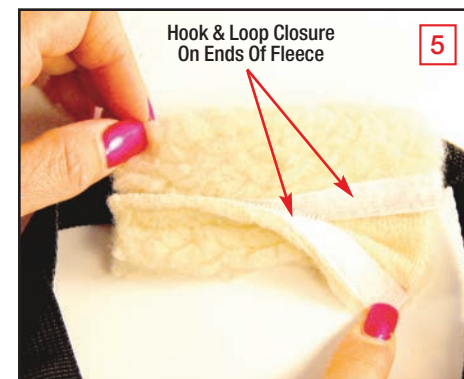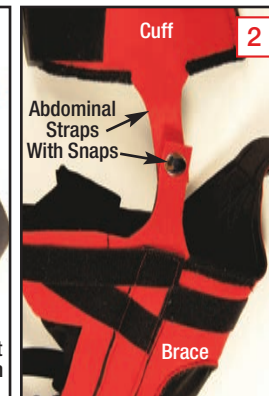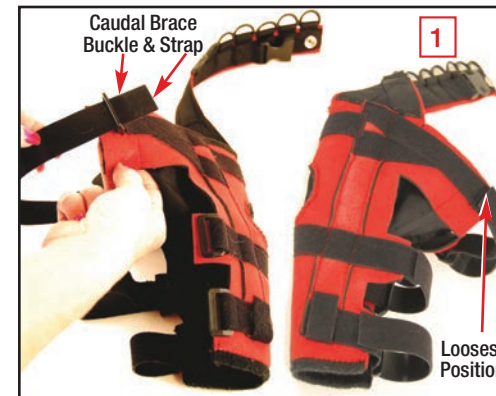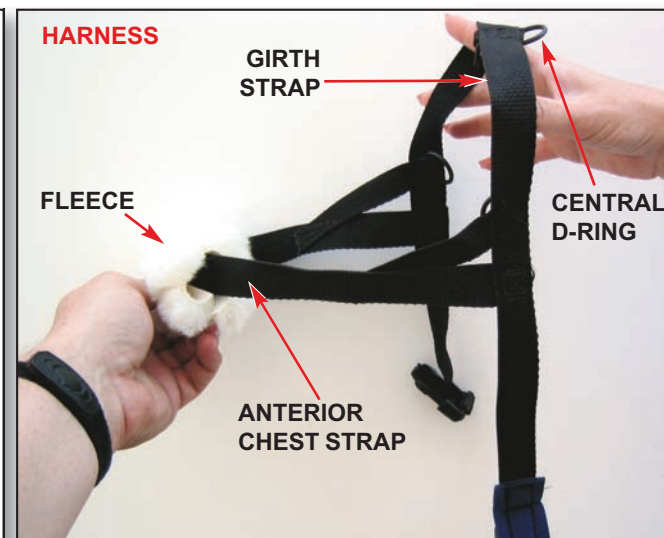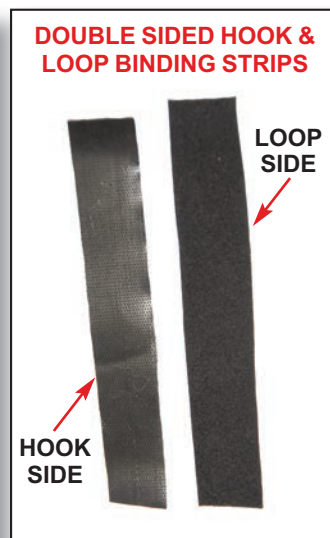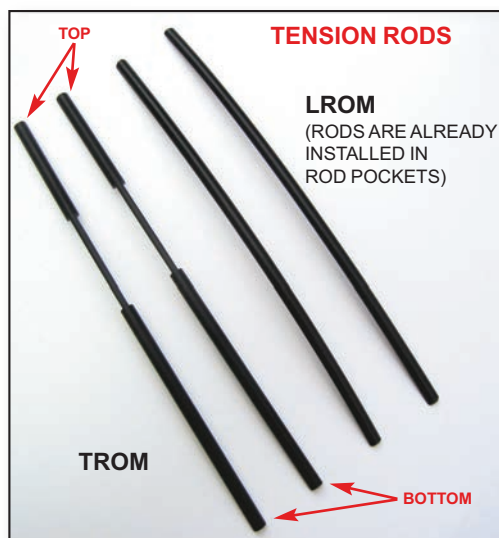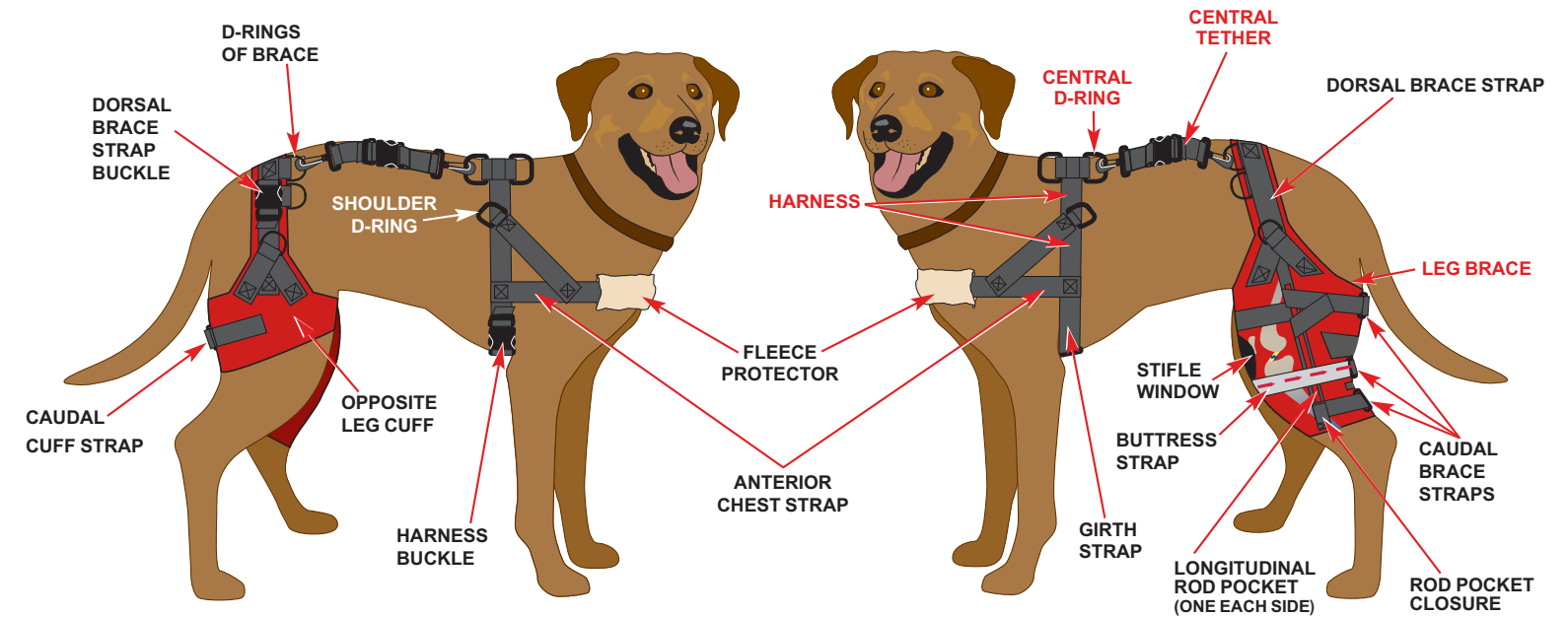

To order by phone, or request product literature from the United States, its principalities or Canada, please use our toll free number: **800-443-4055**

Outside the United States please call **847-634-1700**

For Internet orders, or product information, please see our web site at www.woundwear.com

For assistance with our products, call: **847-634-1700**

